



# Airway Suction Trainer

## Quick Start Guide

The Airway Suction Trainer is used for hands-on training with oral, nasal, and tracheostomy suctioning techniques. Use this lifelike model to practice the insertion of suction catheters into the nasal cavity, oral cavity, and tracheostomy site.



### Product includes:

- Airway Suction Trainer
- 20 lubricant packets
- Oral suction handle
- Nasal suction catheter
- Tracheostomy suction catheter
- 1-year limited warranty
- Online access to curriculum


**NOTICE! To access your curriculum, visit:**

<https://www.realityworks.com/airwaysuctiontrainer-downloads>

# AIRWAY SUCTION TRAINER


## Parts identification:

- 1. Airway Suction Trainer
- 2. Removable face half
- 3. Transparent plate
- 4. Bronchus with clasp
- 5. Oral suction handle
- 6. Nasal suction catheter
- 7. Tracheostomy suction catheter
- 8. Lubricant packet

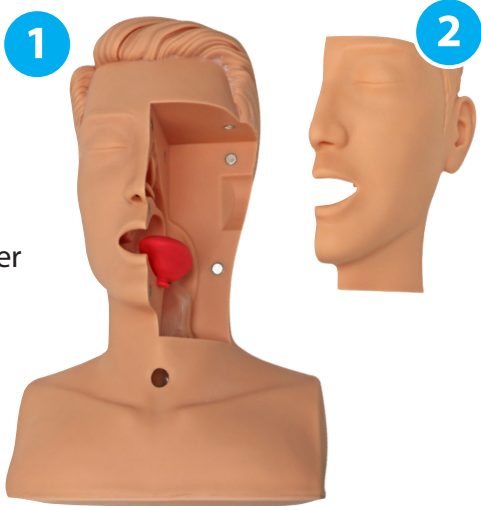

## Assembly instructions:

1. Gather the trainer, bronchus, and clasp

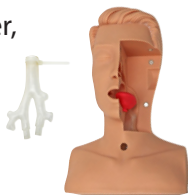

2. Slide the bronchus onto the fitting




3. Slide the bronchus clasp up around the fitting and tighten



4. Attach the transparent plate by aligning the magnets and pressing the plate into the side of the trainer



5. Attach the face half by aligning the magnets and pressing the face half into the side of the trainer

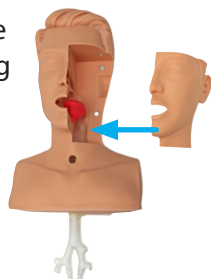

## Care and Cleaning instructions:

1. Release the bronchus clasp, then grab the bronchus and pull it straight out from the bottom of the trainer.
2. Bring the bronchus to a sink and fill it with hot water.
3. Plug the top of the bronchus with your thumb and swish the water around, then dump it out. Repeat this 5 times. Use a brush on the bronchus if needed.
4. Clean the esophagus from the bottom of the trainer with a brush as needed. **NOTE:** Do not immerse the simulator itself in fluids of any kind.
5. Wipe the simulator skin, mouth, and nose areas with a slightly damp cloth, then pat it dry with a clean cloth. Allow all parts to dry overnight.
6. Reattach the dry parts by following the assembly instructions above.
7. Store the simulator in a dry and warm place out of direct sunlight.
8. Only use the simulator on a clean surface. Avoid ink pens, felt-tip markers, acetone, iodine, or other staining products. Do not place the simulator on newsprint or inked lines of any kind.

# NEED MORE HELP?

## Additional help and information

### Online Product Support

We're here to help! Our online Product Support features video tutorials, downloadable documents, frequently asked questions and product notices, all of which make it easy for you to get support for products, software and more, day or night. Find our online product support area here: <https://www.realityworks.com/knowledge-base/>

### Online Store

Supplies, accessories, select simulators, curriculum and more are available for purchase from our online store 24 hours a day!