



Naloxone Training Kit

Instructor Guide

Use the Naloxone Training Kit and included curriculum to teach what an opioid overdose looks like as well as proper procedures for treatment with naloxone through the nasal passage and direct injection.

Product Includes:

- 4 naloxone intranasal trainers (reusable)
- Large Injection Pad
- Ampule breaker
- 10 filter needles
- Naloxone Injection Supply

Pack includes:

- 10 3 cc syringes
 - 10 safety needles
 - 10 simulation naloxone ampules
 - 10 cloth bandages
 - 20 alcohol pads
- Storage case
 - Curriculum
 - 1-year limited warranty



ATTENTION!

- It is recommended to use water or saline for injection practice; other fluids may stain the product.
- Practicing injections without fluid may cause damage to the injection pad; using water or saline will prolong the life of the product.

Cleaning instructions for the Large Injection Pad

1. After use, separate layers of skin
2. Wash parts with mild soap and a soft cloth
3. Rinse well using warm water
4. Squeeze or wring out the additional water from all foam pieces
5. Allow to dry overnight

NALOXONE TRAINING KIT

Parts identification:

1. Naloxone intranasal trainer
2. Large Injection Pad
3. 3 cc syringe
4. Safety needle
5. Ampule breaker
6. Filter needle
7. Naloxone ampule
8. Cloth bandage
9. Alcohol pad



NOTE: For educational purposes only, not to be used on humans