



Epinephrine Training Kit Instructor Guide

Use the Epinephrine Training Kit and included curriculum to teach what an anaphylaxis reaction looks like as well as how to properly treat with auto-injector pens and direct injection.

Product Includes:

- 10 EpiPen® trainers (reusable)
- Large Injection Pad
- 2 epinephrine ampules
- Ampule breaker
- 2 glide filter needles
- 2 glide needles
- Epinephrine Injection

Supply Pack includes:

- 10 1 cc syringes
- 10 cloth bandages
- 10 safety needles
- 20 alcohol pads
- 10 simulation epinephrine vials
- Storage case
- Curriculum
- 1-year limited warranty



NOTICE! To access your curriculum, visit:

www.realityworks.com/epinephrinetrainingkit-downloads

ATTENTION!

- It is recommended to use water or saline for injection practice; other fluids may stain the product.
- Practicing injections without fluid may cause damage to the injection pad; using water or saline will prolong the life of the product.



Cleaning instructions for the Large Injection Pad

1. After use, separate layers of skin
2. Wash parts with mild soap and a soft cloth
3. Rinse well using warm water
4. Squeeze or wring out the additional water from all foam pieces
5. Allow to dry overnight

EPINEPHRINE TRAINING KIT

Parts identification:

1. EpiPen® trainer
2. Large Injection Pad
3. Epinephrine ampule
4. Ampule breaker
5. Glide filter needle
6. Glide needle
7. 1 cc syringe
8. Safety needle
9. Simulation epinephrine vial
10. Cloth bandage
11. Alcohol pad

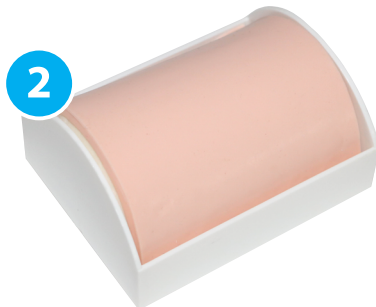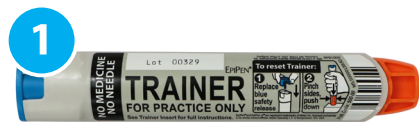

NOTE: For educational purposes only, not to be used on humans