



Canine Vet Trainer Quick Start Guide

Use the realistic Canine Vet Trainer for consistent, hands-on practice caring for a male dog. Participants can practice venipuncture, catheterization, bladder and lymph node palpation, and subcutaneous and intramuscular injections.



Note: IV bag style may vary.

Product includes:

- Canine Vet Trainer
- Dog collar with tag
- IV stand with IV bag
- Pressure bulb
- 3 palpation bladders
- Syringe with 22-gauge needle
- 60-mL syringe
- 8-Fr Catheter
- 8 lymph node markers
- Carrying bag
- Downloadable curriculum
- 1-year limited warranty


NOTICE! To access your curriculum, visit:

WARNING: This is not a toy!

Parts identification:

1. IV stand with bag
2. Pressure bulb*
- *Bulb style may vary
3. 60-mL syringe
4. Full bladder
5. Empty bladder
6. Stones bladder
7. Dog collar with tag
8. Syringe with 22-gauge needle
9. Catheter
10. Lymph node markers

Note: There is a dry shampoo powder applied to the bladder to reduce friction; this may wear off with frequent use.

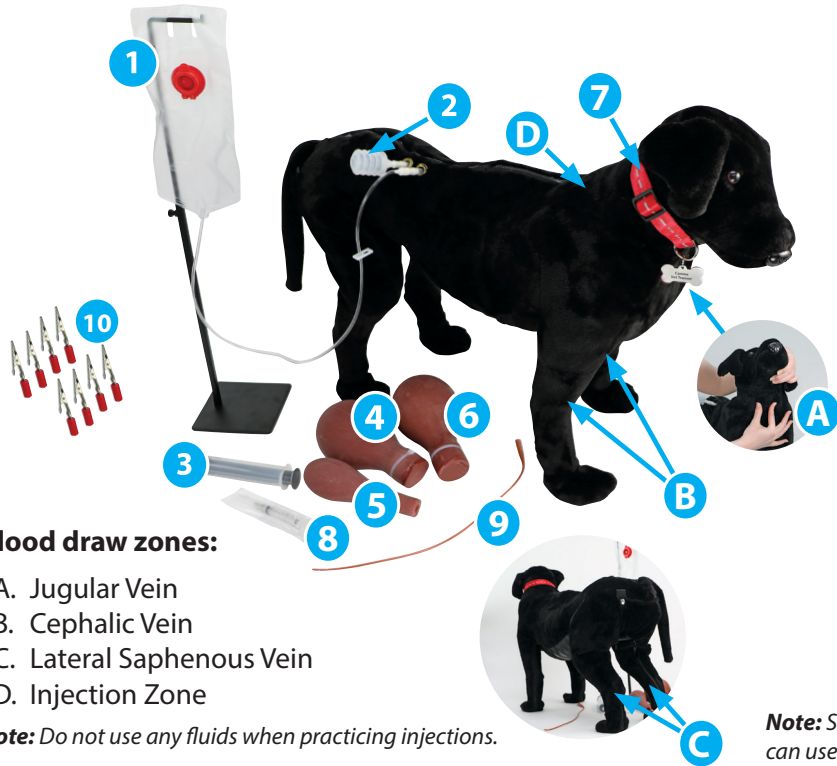


Blood draw zones:

- A. Jugular Vein
- B. Cephalic Vein
- C. Lateral Saphenous Vein
- D. Injection Zone

Note: Do not use any fluids when practicing injections.

Lymph nodes

- Mandibular**
- Popliteal**
- Prescapular
- Axillary
- Inguinal

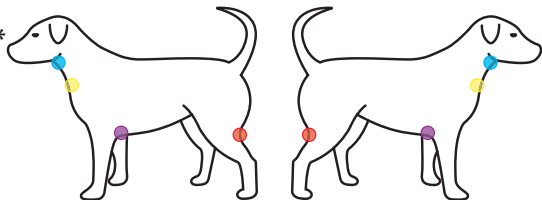


Note: Students can use the lymph node markers to show where each lymph node is located. Open the lymph node markers before removing from the trainer to avoid ripping the hide.

**Students can palpate these swollen lymph nodes on the right side of the trainer.

Assembly Instructions:

Assembling the IV stand


1. Slide the rod with the hook into the metal tube and secure it in place with the set screw on the side of the metal tube. Make sure the rod is extended to its maximum height.
2. Remove the screw from the bottom of the rod and use it to secure the rod to the base.

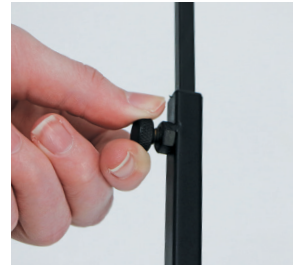

Set-up instructions:

Insert bladders

1. Unsnap the buttons by the trainer's tail and place a bladder inside the trainer with the bladder's rounded side facing away from you.
2. Make sure the bladder sits just above or in front of the penis, and snap the buttons by the tail back together.

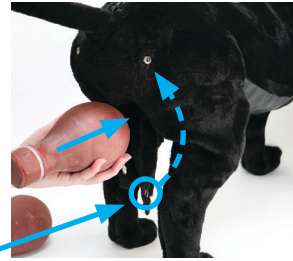


Note: Do not use any fluids when practicing catheterization.

Prepare for blood draw

1. Fill the IV bag with water and hang it on the stand.

Note: You can add red food coloring to the water to make it look more like blood.

2. Connect the luer lock on the IV bag to the corresponding tube sticking out of the trainer. Make sure the bag is as far above the trainer as possible.

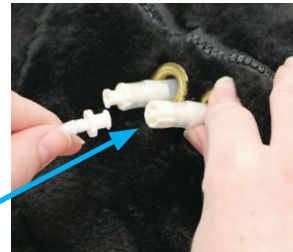

3. Once water starts to come out of the other tube, use the 60-mL syringe to pull the water out and push it back in 5-10 times or until you no longer see any big air bubbles in the IV bag.



Note: If you're still seeing bubbles big air bubbles in the IV bag, you can also fill a cup with water and inject that into the trainer.



CANINE VET TRAINER

4. Optional: Replace the syringe with the pressure bulb.
5. Once the trainer is full, pinch off the IV tube with the attached plastic piece.
6. Apply light pressure to the pressure bulb or syringe so you can feel the veins and release pressure when you're ready to draw blood.

Note: The veins are made from a latex material wrapped in taping.

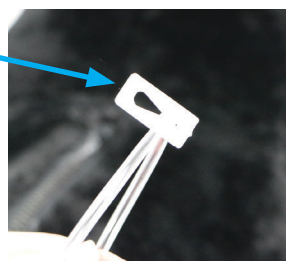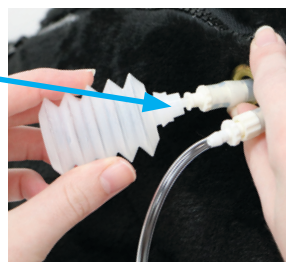

Cleaning instructions:

- Spot clean the fur with water and mild soap.
Do not remove the hide. Let it air dry.
- Flush out the tubing using the 60-mL syringe.

Additional help and information

Online Product Support

We're here to help! Our online Product Support features video tutorials, downloadable documents, frequently asked questions, and product notices, all of which make it easy for you to get support for products, software, and more day or night. Find our online product support area here:

www.realityworks.com/knowledge-base

Online Store

Supplies, accessories, select simulators, curriculum, and more are available for purchase from our online store 24 hours a day!

Find our online store here: www.realityworks.com