



Dementia Simulation Experience Quick Start Guide

The Dementia Simulation Experience and included lessons and activities help participants experience what it is like to have dementia.



Product includes:

- Dementia Simulation Experience (10 student kits)
 - Dementia simulation glasses
 - Headphones and MP3 player
 - 10 dementia simulation audio files
 - Gloves
 - Storage case
- Soft-sided storage case
- 1-year limited warranty
- Downloadable curriculum




NOTICE! To access your curriculum, visit:

www.realityworks.com/dementiasimulationexperience-downloads

MP3 player parts identification:

- A. Power switch
- B. Micro SD slot
- C. Headphone jack
- D. USB charging port
- E. Play/pause button
- F. Menu
- G. Skip track
- H. Volume control

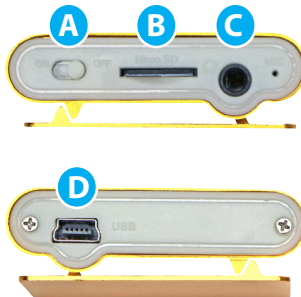

DEMENTIA SIMULATION EXPERIENCE


Instructions:

Charging

1. Plug the charging cord into a power block or USB charging port.
2. Plug the mini USB into the MP3 player.

Playing Tracks

1. Plug the headphones into the headphone port on the MP3 player.
2. Set the power switch of the MP3 player to on, then press **M** to access the tracks.
3. Use the **Skip Track** buttons to cycle through tracks until you find the appropriate one for your scenario.
4. Set the power switch of the MP3 player to off when finished.

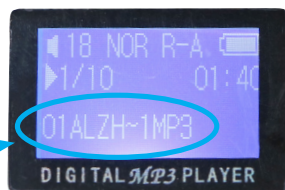

Cleaning instructions:

- Gloves may be machine washed and dried as desired

Additional help and information

Online Product Support

We're here to help! Our online Product Support features video tutorials, downloadable documents, frequently asked questions, and product notices, all of which make it easy for you to get support for products, software and more, day or night. Find our online product support area here:

<https://www.realityworks.com/knowledge-base>

Online Store

Supplies, accessories, select simulators, curriculum, and more are available for purchase from our online store 24 hours a day!

Find our online store here: www.realityworks.com