



Feline Vet Trainer Quick Start Guide

This realistic Feline Vet Trainer provides consistent, hands-on practice with several key cat care skills. This male cat features movable limbs for practicing restraints and positioning during a variety of procedures. Additionally, students can practice feline CPR, venipuncture, injections, blood draw, and catheterization.

Parts identification:

1. Feline Vet Trainer
 2. IV bag and stand
 3. Syringe with 20-gauge needle
 4. Luer lock syringe
 5. 3.5 Fr catheter
 6. 100 CPR bags
 7. 2 CPR bag replacement hooks
 8. Carrying bag
- Downloadable curriculum
 - 1-year limited warranty




NOTICE! To access your curriculum, visit:
www.realityworks.com/felinevettrainer-downloads

WARNING: This trainer is for educational use only! It is not a toy.

FELINE VET TRAINER

Blood draw zones:


- A. Jugular vein
- B. Cephalic vein
- C. Femoral vein

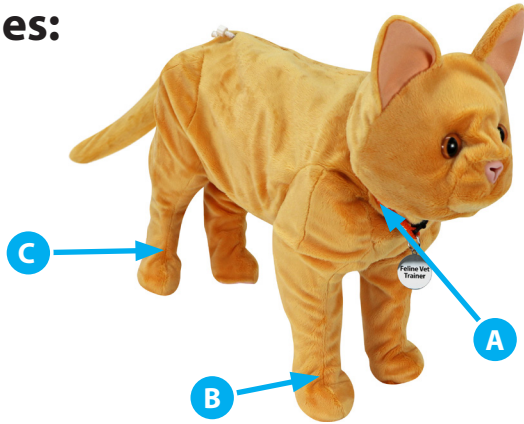

Injection sites:

- D. Pilot Study Site
- E. FeLV
- F. Rabies
- G. FVRCP

Note: Do not use fluids during injection practice.

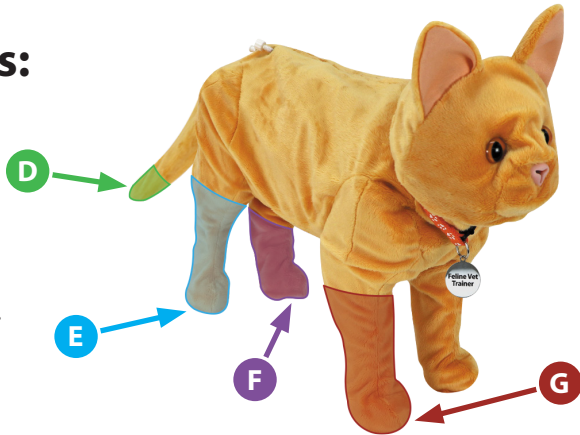

Assembly Instructions:

Assembling the IV bag and stand

1. Slide the rod with the hook into the metal tube and secure it in place with the set screw located on the side of the metal tube. Make sure the rod is extended to its maximum height.
2. Remove the screw from the bottom of the rod and use it to secure the rod to the base.



Set-up instructions:

Preparing for blood draw

1. Pinch the IV tubes closed with the attached clamps and fill the bag with water.
- Note:** *Keep the water level below the IV bag port.*
2. Hang the IV bag on the stand and connect the bag's Luer locks to the corresponding tubes on the trainer.
3. Adjust the clamps to open the IV tubes and make sure the bag is as far above the trainer as possible.

Note: *The veins are made from a latex material wrapped in tape.*



Performing for mouth-to-snout CPR

1. Lay the trainer on its right side and unzip the spine zipper.
2. Attach a CPR bag to the CPR bag hook.
3. Starting with the opposite end of the hook, feed the hook through the trainer's nose.
4. Pull the hook through the tube near the trainer's front leg. Stop when the face shield part of the bag reaches the trainer's snout.
5. Spread the face shield over the trainer's snout and detach the hook from the bag.
6. Zip the trainer back up and practice mouth-to-snout CPR. Breathing into the bag will simulate lung inflation and deflation.
7. Remove the CPR bag from the trainer by pulling it completely through the trainer with your hand.

Note: *You can also practice chest compressions with the Feline Vet Trainer.*



Cleaning instructions:

- Spot clean the fur with water and mild soap. Do not remove the hide. Let it air dry.
 - Flush out the tubing using the Luer lock syringe.
-

Additional help and information

Online Product Support

We're here to help! Our online Product Support features video tutorials, downloadable documents, frequently asked questions, and product notices, all of which make it easy for you to get support for products, software and more, day or night. Find our online product support area here: www.realityworks.com/knowledge-base

Online Store

Supplies, accessories, select simulators, curricula, and more are available for purchase from our online store 24 hours a day! Find our online store here: www.realityworks.com