



Canine CPR Trainer

Quick Start Guide

This realistic Canine CPR Trainer provides consistent, hands-on practice with mouth-to-snout CPR, tracheal intubation with a Ambu bag, and femoral artery pulse palpation.

Parts identification:

1. Canine CPR Trainer
 2. 50 CPR bags
 3. 2 CPR bag replacement hooks
 4. Ambu bag
 5. Endotracheal tube
 6. Pulse bulb
 7. 10-ml syringe
 8. Carrying bag
- Downloadable curriculum
 - 1-year limited warranty



WARNING: This trainer is for educational use only. It is not a toy.

CANINE CPR TRAINER

Setup instructions:

Preparing for mouth-to-snout CPR practice

1. Lift the trainer's hide cover and slide the CPR bag replacement hook through the chest cavity until you can see the hook exit the trainer's nose. If you're having difficulty getting the hook through, dip it in a small cup of water to lubricate.
2. Attach the CPR bag to the hook and slowly pull the hook back out through the chest cavity, folding the bag as you feed it through the nose.




3. Stop pulling when the bag's face shield reaches the trainer's snout and detach the hook from the bag.
4. Spread the face shield over the trainer's snout and cover the bag with the trainer's hide cover.
5. Practice mouth-to-snout CPR. Breathing into the bag will simulate lung inflation and deflation.
6. When practice is complete, remove the CPR bag by slowly pulling it through the trainer's chest cavity with your hand.



Note: You can also practice chest compressions with the Canine CPR Trainer.

Preparing for tracheal intubation practice

1. Ensure the cuff on the endotracheal tube is deflated and feed the tube through the opening in the back of the trainer's mouth until you can no longer see the cuff.
2. Attach the syringe to the pilot balloon and plunge the syringe to fill the cuff with air.
3. Detach the syringe and repeat step 2.
4. Detach the syringe and attach the Ambu bag to the endotracheal tube.
5. Squeeze the bag to see the trainer's chest rise and fall.

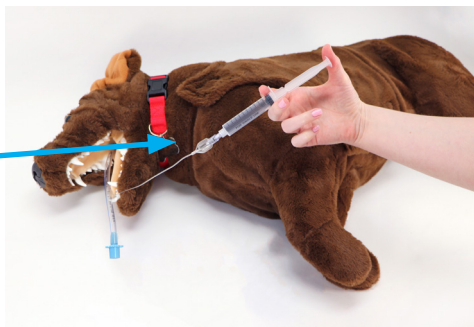

Preparing for femoral artery pulse palpation practice

1. Attach the pulse bulb to the femoral artery tube in the trainer's hind leg and squeeze the bulb to simulate the pulse.



Cleaning instructions:

- Spot clean the fur with water and mild soap. Let it air dry.

Additional help and information

Online Product Support

We're here to help! Our online Product Support features video tutorials, downloadable documents, frequently asked questions, and product notices, all of which make it easy for you to get support for products, software and more, day or night. Find our online product support area here: www.realityworks.com/knowledge-base

Online Store

Supplies, accessories, select simulators, curricula, and more are available for purchase from our online store 24 hours a day! Find our online store here: www.realityworks.com