




Canine Intubation Simulator

Quick Start Guide

This Canine Intubation Simulator provides consistent, hands-on practice with canine intubation.

Parts identification:

1. Canine Intubation Simulator
 2. Endotracheal tube
 3. Connector cord
 4. Power cord
 5. Control box
- Downloadable curriculum
 - 1-year limited warranty

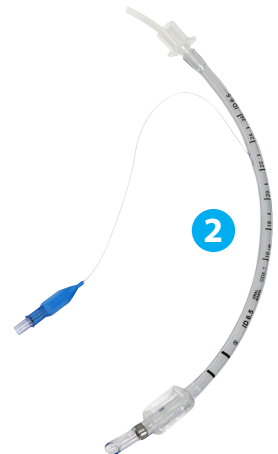

WARNING: This simulator is for education use only! It is not a toy.

CANINE INTUBATION SIMULATOR

Set-up instructions:


1. Insert the power plug on the power cord into an outlet and plug the cord's DC connector into the control box.




2. Plug the connector cord into the control box, securing it in place with the screws.





3. Plug the other end of the connector cord into the simulator, securing it in place with the screws.

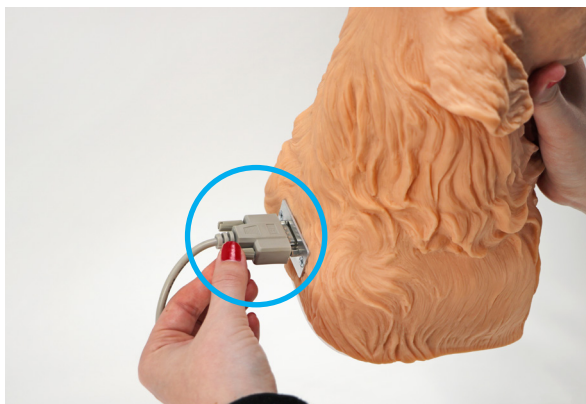

4. Press the power button on the control box. The Power Supply State light turns on.

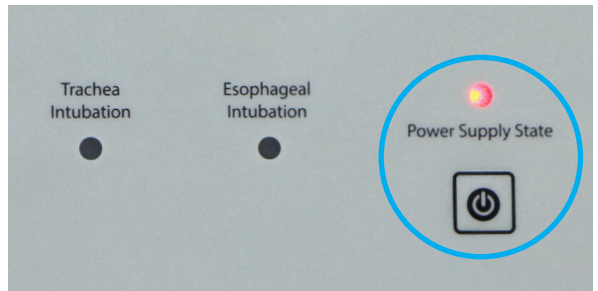

5. Intubate the simulator with the endotracheal tube. If you intubate correctly, the Trachea Intubation light turns on. If you intubate incorrectly, the Esophageal Intubation light turns on.

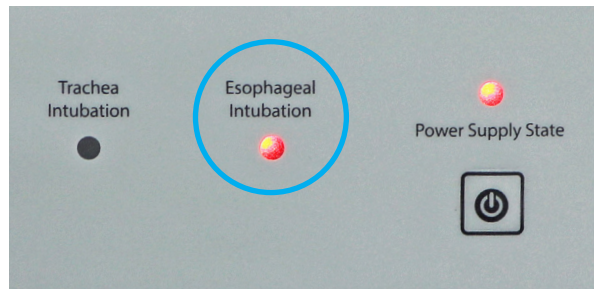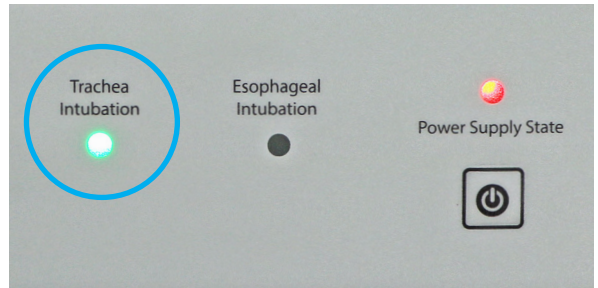

CANINE INTUBATION SIMULATOR

Cleaning instructions:

- Wipe down simulator, tube, and control box with a disinfectant wipe and air dry.
-

Additional help and information

Online Product Support

We're here to help! Our online Product Support features video tutorials, downloadable documents, frequently asked questions, and product notices, all of which make it easy for you to get support for products, software and more, day or night. Find our online product support area here: www.realityworks.com/knowledge-base

Online Store

Supplies, accessories, select simulators, curricula, and more are available for purchase from our online store 24 hours a day! Find our online store here: www.realityworks.com