




Canine Venipuncture Legs Quick Start Guide

The Canine Venipuncture Legs help students learn proper canine venipuncture techniques and why this procedure is important.

Product includes:

- A. 3 Large Venipuncture Legs
- B. 3 Small Venipuncture Legs
- C. 6 IV stands
- D. 6 IV bags
- E. 6 disposable syringes with needles
- Downloadable curriculum
- 1-year limited warranty

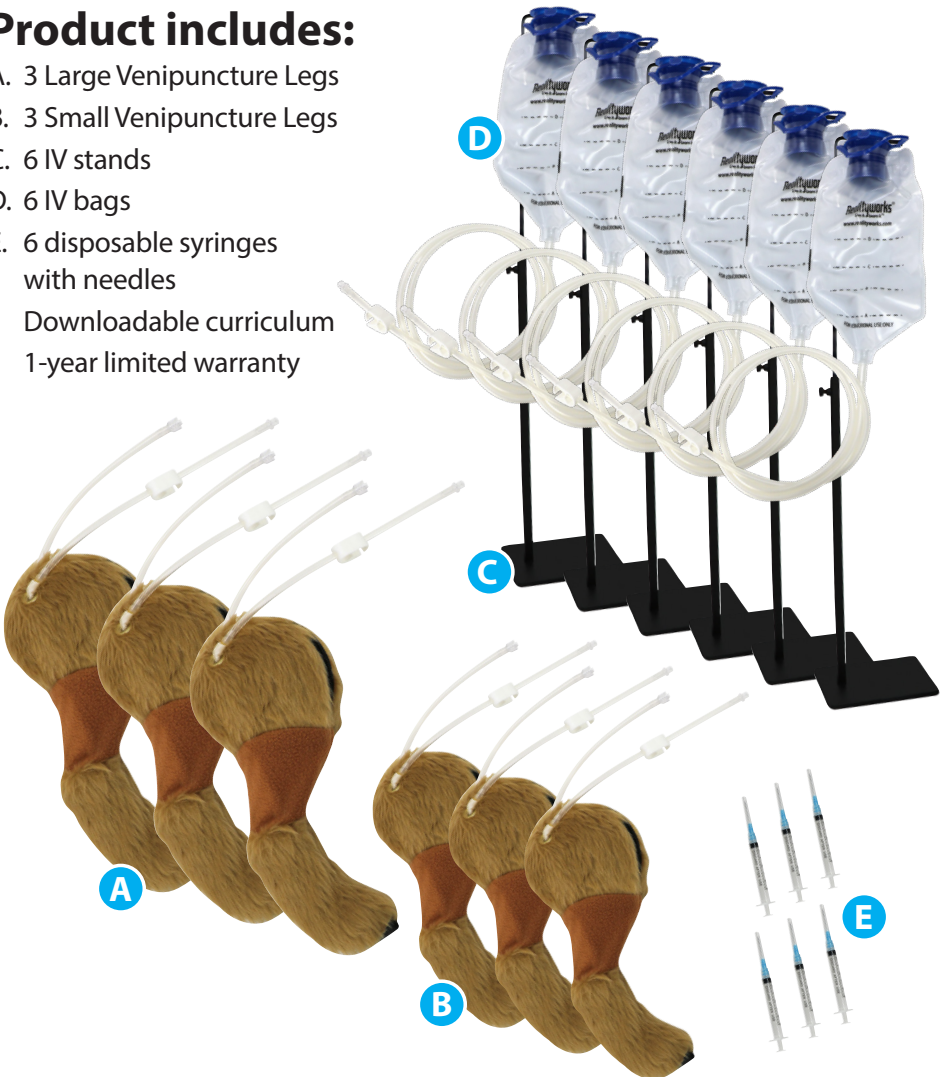

CANINE VENIPUNCTURE LEGS

Set-up instructions:


Assembling the IV stands

1. Slide the rod with the hook into the metal tube and secure it in place with the set screw on the side of the metal tube. Make sure the rod is extended to its maximum height.
2. Remove the screw from the bottom of the metal tube and use it to secure the tube to the base.




Preparing for venipuncture

1. Close the IV bag's tube with the clamp and fill the IV bag with water.

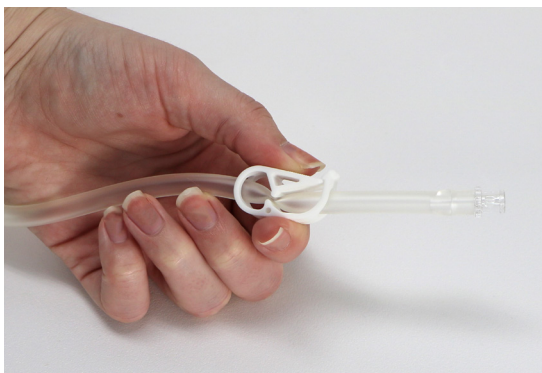

2. Hang the IV bag on the stand and connect the Luer lock on the IV bag to the corresponding tube on the Canine Venipuncture Leg.

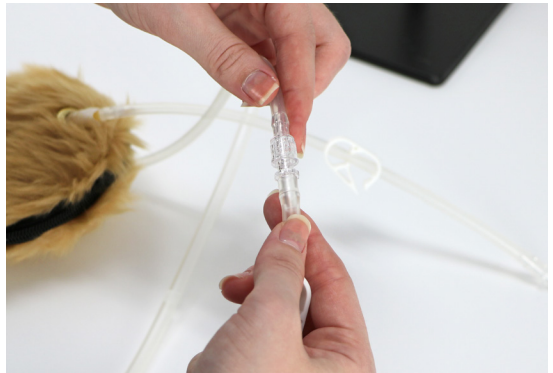

3. Unclamp the IV bag's tube, allowing water to flow into the Canine Venipuncture Leg.
4. Once the Canine Venipuncture Leg is full, close its tube with the clamp.

CANINE VENIPUNCTURE LEGS

Cleaning instructions:

- Spot clean fur with water and mild soap
 - Flush out tubing with the syringe
-

Additional help and information

Online Product Support

We're here to help! Our online Product Support features video tutorials, downloadable documents, frequently asked questions, and product notices, all of which make it easy for you to get support for products, software and more, day or night. Find our online product support area here: www.realityworks.com/knowledge-base

Online Store

Supplies, accessories, select simulators, curricula, and more are available for purchase from our online store 24 hours a day! Find our online store here: www.realityworks.com