



Venipuncture Training Arms Quick Start Guide

The realistic Venipuncture Training Arms help students practice phlebotomy, IV starts, and intramuscular injections.

Product includes:

- A. 3 dark-skinned training arms
- B. 3 light-skinned training arms
- C. 6 IV stands
- D. 6 IV bags
- Downloadable curriculum
- 1-year limited warranty



VENIPUNCTURE TRAINING ARMS

Set-up instructions:


Assembling the IV stands

1. Slide the rod with the hook into the metal tube and secure it in place with the set screw on the side of the metal tube. Make sure the rod is extended to its maximum height.
2. Remove the screw from the bottom of the metal tube and use it to secure the tube to the base.



Preparing for venipuncture

1. Close the IV bag's tube with the clamp and fill the bag with water.

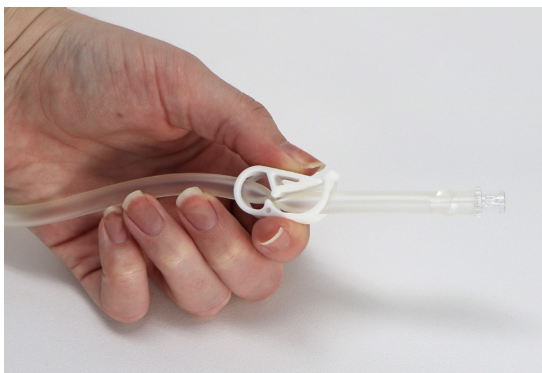

2. Hang the IV bag on the stand and connect the Luer lock on the bag to the corresponding tube on the Venipuncture Training Arm.



3. Unclamp the IV bag's tube, allowing water to flow into the Venipuncture Training Arm.
4. Once the Venipuncture Training Arm is full, close its tube with the clamp.

VENIPUNCTURE TRAINING ARMS

Cleaning instructions:

- Wipe the arm with warm water. Pat dry with a clean cloth.
 - Empty the IV bag and all tubing. Allow to dry overnight.
-

Additional help and information

Online Product Support

We're here to help! Our online Product Support features video tutorials, downloadable documents, frequently asked questions, and product notices, all of which make it easy for you to get support for products, software and more, day or night. Find our online product support area here: www.realityworks.com/knowledge-base

Online Store

Supplies, accessories, select simulators, curricula, and more are available for purchase from our online store 24 hours a day! Find our online store here: www.realityworks.com