



Canine Vital Signs Trainer

Quick Start Guide

The Canine Vital Signs Trainer is a hands-on teaching tool for practicing basic dog health assessments. Students can measure heart rate, respiratory rate, and temperature; collect sample specimens; practice safe restraint techniques; and respond to real-world vital signs scenarios.

Product Includes:

- A. Canine Vital Signs Trainer
- B. Thermometer*
- C. Simulated Stool
- D. Fecal loop
- E. (2) 60-mL syringes
- F. Urine collector
- G. Collection container
- H. Cleaning tool
- I. Accessories storage box
 - 5 thermometer sleeves
 - Storage bag
 - Downloadable curriculum
 - 1-year limited warranty



*Thermometer appearance may vary

NOTICE! To access your curriculum, visit:

www.realityworks.com/caninevitalsigns-downloads


NOTE: This is not a CPR trainer. Do not attempt to perform chest compressions on this model.

CANINE VITAL SIGNS TRAINER

Parts Identification:


Trainer

1. Power switch
2. Scenario display
3. Previous scenario button
4. Next scenario button

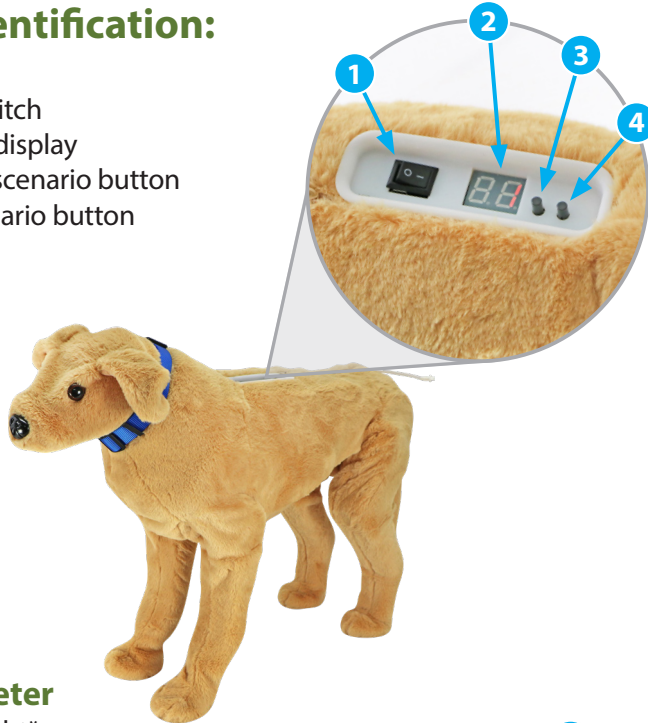

Thermometer

5. Pairing light*
6. Battery power light
7. Fahrenheit/Celsius button
8. Power switch
9. Pairing button*

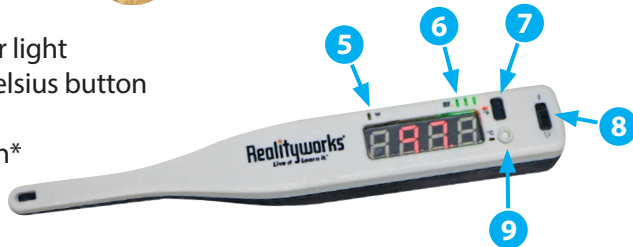

*This product includes a Bluetooth button and indicator light as part of its standard design. However, Bluetooth functionality is not available in this model.

Use Instructions:

Heart Rate, Respiratory Rate, and Rectal Temperature Simulation

1. Flip the power switch on the simulator's back and click the buttons by the display to cycle through scenarios.



2. To take the trainer's rectal temperature, flip the thermometer's power switch and insert the thermometer into the trainer's rectum.



CANINE VITAL SIGNS TRAINER

Scenario Key

Heart and Respiratory Rate Scenarios

Scenario Number	Heart Beats Per Minute	Breaths Per Minute
1	Healthy: 75 bpm	Healthy: 18 bpm
2	Healthy: 90 bpm	Healthy: 25 bpm
3	High: 125 bpm	Fast: 40 bpm
4	High: 140 bpm	Fast: 40 bpm
5	Low: 30 bpm	Slow: 5 bpm
6	Low: 45 bpm	Slow: 5 bpm

Temperature Scenarios

Scenario Number	Temperature (Fahrenheit)
7	Healthy: 99.5-100.5
8	Healthy: 101-102
9	High (Fever): 103-104
10	High (Fever): 104.5-105.5
11	Low: 97-98

Urine Collection Simulation

1. Fill the syringe with the Luer lock tip with water and secure it to the tube near the trainer's tail.
2. Gently depress the plunger and use the urine collector to catch the water.



NOTE: You can mix food coloring in the water for added realism.

CANINE VITAL SIGNS TRAINER

Fecal Collection Simulation

1. Follow the instructions on the Simulated Stool package to mix up the fecal matter and use the syringe with the Toomey tip to apply the simulated feces to the trainer's rectum.
2. Use the fecal loop to collect a sample.




Cleaning Instructions:

- Spot clean the fur with water. Do not remove the hide. Do not steam clean. Let air dry.
- Flush out the tubing with the Luer-lock-tip syringe.
- Pull the rectum out of the trainer. Use water, the cleaning tool, and the Toomey-tip syringe to clean it. Let air dry.

NOTE: It may take some maneuvering to get the rectum back in place.

Low Battery Warnings

- The thermometer has LED indicators for battery life. They will flash when the battery is too low to function and no temperature will display. Open the battery compartment on the back of the thermometer with a Phillips head screwdriver and replace the AAA battery.

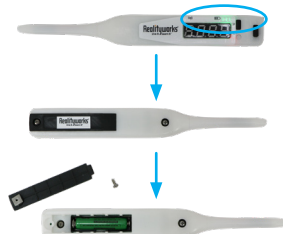


Low Battery Warnings

- The Canine Vital Signs Trainer will show "88" on the scenario display when the battery is too low and will cease to function. Unzip the trainer and replace the 4 C batteries inside.



Downloads

We want to make sure you have the tools you need to get started in your classroom. Our Downloads page houses this simulator's curriculum, slides, and completion certificate.

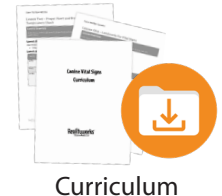

Access your downloadable resources here:

www.realityworks.com/caninevitalsigns-downloads

CANINE VITAL SIGNS TRAINER

Additional Help and Information

We're here to help! Our online Product Support features video tutorials, downloadable documents, frequently asked questions, and product notices, all of which make it easy for you to get support for products, software and more, day or night. Find our online product support area here: www.realityworks.com/knowledge-base

Take Learning Further

Thank you for choosing Realityworks! We're committed to providing engaging, hands-on learning tools that enhance student understanding and skill development. Explore our full range of educational solutions designed to support career and technical education, STEM learning, and more.



Visit www.realityworks.com to discover additional products, resources, and support materials to elevate your teaching experience.