



Medication Administration Trainer

Quick Start Guide

The Medication Administration Trainer is a versatile, life-size torso model designed to help future healthcare professionals build confidence and competence in delivering medications safely and accurately. This realistic trainer allows students to practice administering medications through a variety of routes, including oral, nasal, ophthalmic, otic, topical, rectal, vaginal, and nasogastric.

Product Includes:

- A. Medication Administration Trainer with nasogastric tube
- B. 2 droppers
- C. 5 simulated suppositories
 - Downloadable curriculum
 - 1-year limited warranty



WARNING: For educational use only.

MEDICATION ADMINISTRATION TRAINER

Simulated Medication Administration Instructions:

Otic

- Fill one of the droppers with water and prop the trainer up on its side. Do not twist its head.
- Using the dropper, dispense the water in the trainer's ear canal.

Ophthalmic

- Fill one of the droppers with water and use it to dispense the water into the trainer's eye.

Nasal

- Fill a nasal spray applicator with water and dispense the water in the trainer's open nostril. Do not put more than 60 mL of liquid in the trainer at a time.

Nasogastric

- Fill a syringe with water and dispense the water into the trainer's attached nasogastric tube. Do not put more than 60 mL of liquid in the trainer at a time.

Oral

- Use water to dispense simulated medication into the trainer's mouth.

Topical

- Apply any topical medication or lotion to the trainer's skin.

Vaginal

- Apply a water-based lubricant to the simulated suppository and insert it into the trainer's vaginal opening.

Rectal

- Suppository: Apply a water-based lubricant to the simulated suppository and insert it into the trainer's rectum.
- Micro-Enema: Administer a micro-enema to the trainer via its rectum using water.

Cleaning and Storage Instructions:

- **Ears:** Remove the trainer's ears and the sponge in the ear compartment. Squeeze out the sponge and let air dry.
- **Eyes:** Remove the trainer's eyes and squeeze them out. Let air dry.
- **Nose and Stomach:** Open the compartment on the trainer's torso and remove the fluid reservoir. Be careful not to spill liquid in the compartment. Tip the reservoir upside down to drain. Let air dry. Ensure the black dot faces up when reinserting reservoir.
- **Mouth:** Remove the trainer's three oral sponges (one under the trainer's tongue and two in the back of its mouth) and squeeze them out. Let air dry.
- **Skin:** Wipe down with a damp cloth and pat dry.
- **Vagina and Rectum:** Remove the trainer's pubic area and wipe it down with a damp cloth. Remove the sponge in the pubic compartment and squeeze it out. Let both items air dry.
- **Storage:** Store the trainer FLAT in its box.



MEDICATION ADMINISTRATION TRAINER

Downloads

We want to make sure you have the tools you need to get started in your classroom. Our Downloads page houses this simulator's curriculum, slides, and completion certificate.



Slide
Presentations



Curriculum



Completion
Certificate

Additional Help and Information

We're here to help! Our online Product Support features video tutorials, downloadable documents, frequently asked questions, and product notices, all of which make it easy for you to get support for products, software and more, day or night. Find our online product support area here:

www.realityworks.com/knowledge-base

Take Learning Further

Thank you for choosing Realityworks! We're committed to providing engaging, hands-on learning tools that enhance student understanding and skill development. Explore our full range of educational solutions designed to support career and technical education, STEM learning, and more.



Visit www.realityworks.com to discover additional products, resources, and support materials to elevate your teaching experience.