



Canine Vet Trainer Vein Replacement Kit

Quick Start Guide

Product includes:


- A. 2 rear leg veins
- B. 2 front leg veins
- C. Gloves

Latex Allergy Notice:

The replacement veins are made of latex. Gloves are included to reduce direct contact during installation. Users with a latex allergy or sensitivity should wear gloves and avoid direct skin contact with the latex veins.



NOTE: Glove style may vary

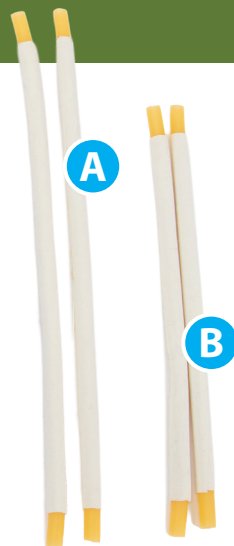

Instructions:

1. Clear a working space for replacement of the front and rear leg veins. The long veins are used in the rear legs, and the short veins are used in the front legs.



2. Lay the Canine Vet Trainer on its side and locate the zipper on the inside of the front leg as shown.

NOTE: It is easiest to work on the leg that is not against the table.



VEIN REPLACEMENT KIT

3. Unzip the zipper and remove the hide from around the internal paw. This is a tight fit so it will take a little effort.



4. Push the loose hide up to expose the vein connection point at the top of the leg.

5. Using a twisting and pulling motion, remove the old veins.



6. Remove both ends of the old front leg veins.

Install the new short vein by twisting and pushing the vein onto the fitting one end at a time. This replacement is now complete.



7. Reverse the procedure to get the hide back into place by starting with pulling the hide back down to the paw.



8. Pull the hide over the paw and zip the zipper back up to the top of the leg.

Flip the Canine Vet Trainer over and repeat these steps on the other front leg.



9. Move to the longer rear leg veins. Start by locating the zipper on the rear leg.

NOTE: It is easiest to work on the leg that is against the table.



10. Unzip the zipper and remove the hide from around the paw.



11. Push the hide up far enough to expose the upper fitting vein connection location as shown.



12. Using a twisting and pulling motion, remove the old veins.

Remove both ends of the old rear leg veins.

Install the long vein on both ends as shown.



VEIN REPLACEMENT KIT

13. Reverse the process by pulling the hide down over the paw.
14. Zip the zipper back up to enclose the veins within the hide.
15. Flip the dog over and repeat this process for the other rear leg vein replacement. The replacement of all 4 leg veins is now complete.



Additional help and information

Online Product Support

We're here to help! Our online Product Support features video tutorials, downloadable documents, frequently asked questions, and product notices, all of which make it easy for you to get support for products, software and more, day or night. Find our online product support area here: www.realityworks.com/knowledge-base

Online Store

Supplies, accessories, select simulators, curriculum, and more are available for purchase from our online store 24 hours a day! Find our online store here: www.realityworks.com