

Chest Tube Management Cleaning Instructions

1. **If using the hemothorax side**, leave the suction and hemothorax control on and disconnect the IV bag to allow water to clear from the simulator until you see bubbles in the water seal of the canister.
2. Turn off the suction control and the hemo/pneumothorax control.
3. Disconnect the bellows to allow any remaining fluid to dry.
4. Disconnect the tubes from the canister and simulator.
5. Remove the plug from the back of the simulator and allow the simulator to dry.
6. Simulator can be wiped down with a disinfectant solution of mild soap and water.