

How do I setup, clean up, or store the Swine Litter Processing Simulator?

Purpose

Setup, clean up and store the Swine Litter Processing Simulators


Prerequisites

- Piglet
- A set of teeth
- Tail cord
- Ear
- Pair of testicles


Steps to Perform

Insert the teeth

1. Prepare the teeth by pressing them against a hard surface and bend them inside.

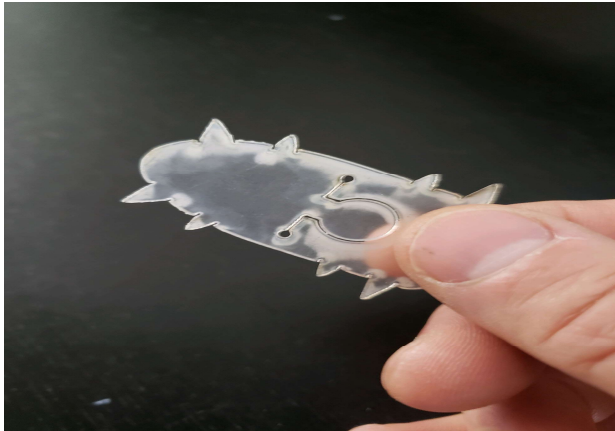

2. Repeat on both sides until all teeth are bent.

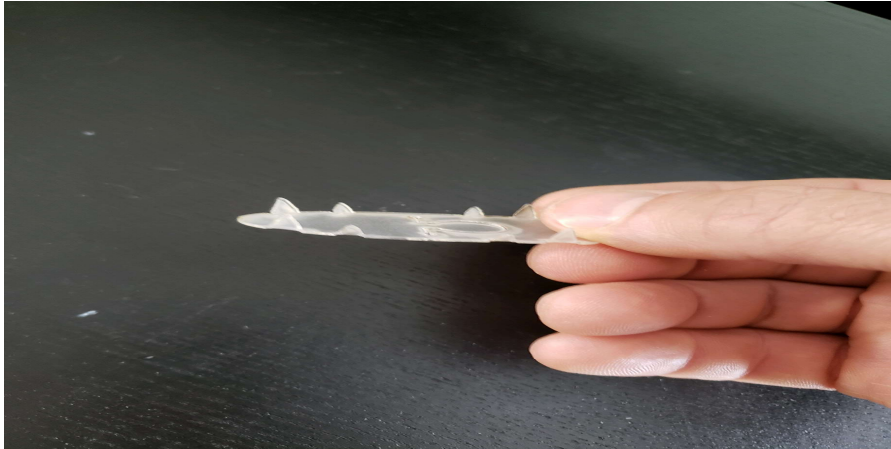

3. Fold the teeth in half to simulate the jaw.



4. Insert the teeth into the piglet's mouth.



Insert the tail

1. Insert the new tail coil in through the back of the piglet.



2. Pull out 2-3 inches of tail.



Insert the ears

1. Curl a set of ears, insert it through one ear, then pull it out the other side until the ear slit fits into the ends of the ear tubes.



Insert the testicles

1. Insert a pair of testicles from the stomach on the piglet.



2. While holding the top of the testicle in the right or left notch, push the testicle down through the fabric to the slot of the castration pouch.



Cleaning Instructions

1. Spot clean the outside of piglet only. Do not submerge in water.
2. Use a soft cloth, soap, and warm water to remove dirt, then remove the soap with a clean, moist cloth.
3. Leave out to air dry, then brush gently.

For additional assistance and video tutorials visit our website:

www.realityworks.com/support/