

Auscultation Student Practice and Assessment Setup

Purpose

Walk you through setting up student practice conditions and assessment conditions.


Access admin main menu

1. Make sure you have the RealityScope turned on, then open the Auscultation Trainer app on the tablet by tapping the icon.
2. When the Auscultation Trainer app opens, tap the star in the Realityworks logo to access the admin menu. The screen will turn from red to blue.
3. If the admin menu does not appear, close the app and try again.




Select Sites and Conditions for Student Practice

1. Open the side menu and select Sites and Conditions.
2. In the Practice tab, you can view all 21 sites and conditions.
3. Select or unselect which conditions you would like the students to practice by tapping on them. The color of the checkmark will change to gray when it is unselected.

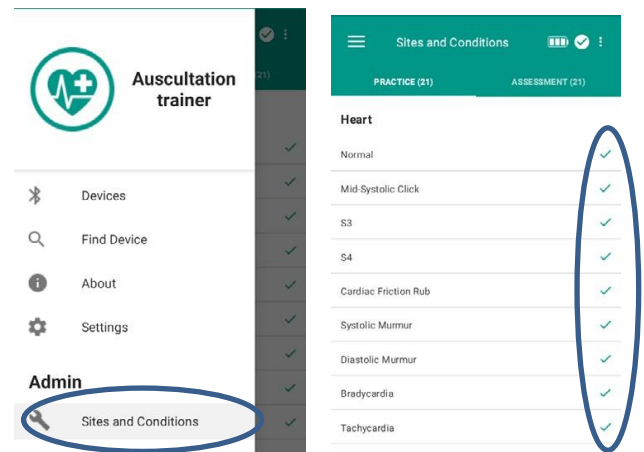

Select Sites and Conditions for Assessment

1. Open the side menu and select Sites and Conditions.
2. In the Assessment tab, select or unselect which conditions you want the students to be assessed on. Unselecting will change the color of the checkmark to gray and move it to the bottom of the list.
3. Drag the conditions to reorder them as desired.

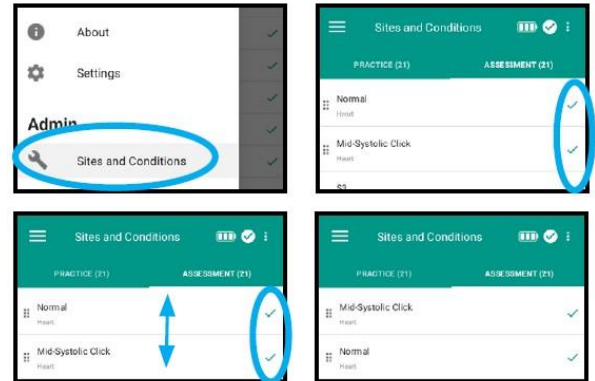

For additional assistance and video tutorials visit our website:

www.realityworks.com/support/