


Bovine Breeder Artificial Insemination Simulator Setup

Purpose

This guide will walk you through setup of your Swine Breeder Artificial Insemination Simulator.

Steps to Inserting the tracts:

1. Remove the Wing nuts from the Tract you are using.
2. Insert the tract into the hole located on the backside of the simulator
3. Align and insert the bolts into the holes located on the inset around the insert hole
4. Press around the outsides of the swine rear end until the tract is flush with the inset hole
5. On the inside of the swine rear end, tighten a wing nut by hand onto each screw
 - a. Note be careful not to overtighten

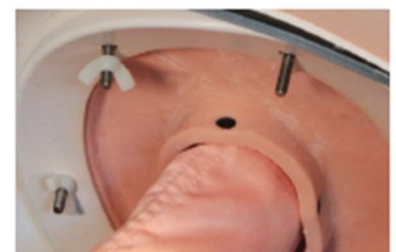

6. For the AI reproductive tract, attach the loops from the tract onto the bolts near the front of the simulator



Steps to perform to insert piglets into the farrowing tract:

Note: It is recommended to liberally apply soapy water (combine water and dish soap) to lubricate the tract and piglets every one or two birthing scenario demonstrations.

Forward presentation position



Double-over position



Backward presentation position



Two pigs presented at once



Breech position

Note: Similar to backward presentation, with back legs tucked under the piglet



For additional assistance and video tutorials visit our website:

www.realityworks.com/support/