




Large Injection Pad Instructor Guide

This Large Injection Pad provides practice for intradermal, intramuscular and subcutaneous injections. Pad features convenient fluid collection tray and has ability to show skin wheal when proper intradermal injection procedure is followed.



Product Includes:

- Large Injection Pad with four layers:
 - Skin
 - Outer Skin
 - Subcutaneous tissue
 - Muscle layer
- Fluid collection tray
- Curriculum
- 1-year limited warranty

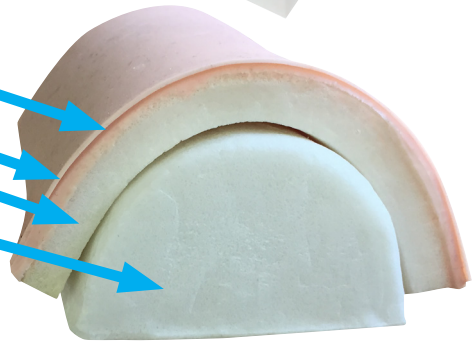

NOTICE! To access your curriculum, visit:


www.realityworks.com/largeinjectionpad-downloads

ATTENTION!

- It is recommended to use water or saline for injection practice; other fluids may stain the product.
- Practicing injections without fluid may cause damage to the injection pad; using water or saline will prolong the life of the product.

Cleaning instructions

1. After use, separate layers of skin.
2. Wash parts with mild soap and a soft cloth.
3. Rinse well using warm water.
4. Squeeze or wring out the additional water from all individual foam pieces.
5. Allow to dry overnight.

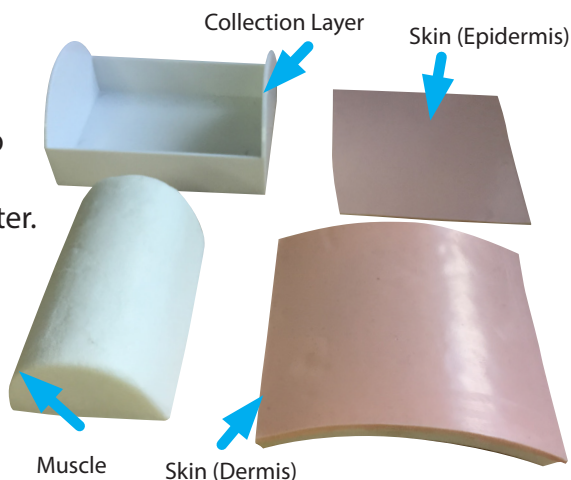

NEED MORE HELP?

Additional Help and Information

Online Product Support

We're here to help! Our online Product Support features video tutorials, downloadable documents, frequently asked questions and product notices, all of which make it easy for you to get support for products, software and more, day or night. Find our online product support area here: www.realityworks.com/support/

Online Store

Supplies, accessories, select simulators, curriculum and more are available for purchase from our online store 24 hours a day! Find our online store here: <http://store.realityworks.com>

Learn about other Realityworks Health Science products online: www.realityworks.com