

Webinar:

Featured Product Demo: Bariatric Nursing Manikin








Realityworks®
Live it. Learn it.™

100% EMPLOYEE-OWNED



Who is Realityworks?

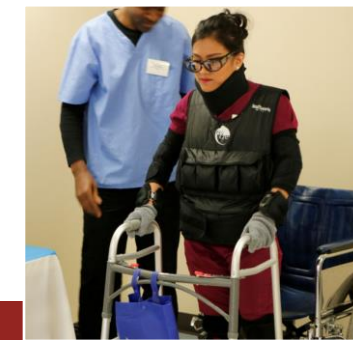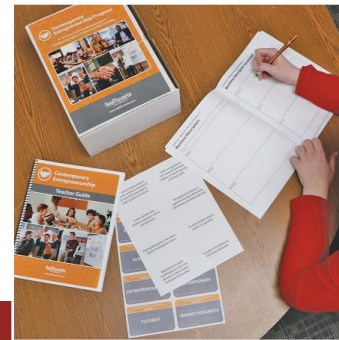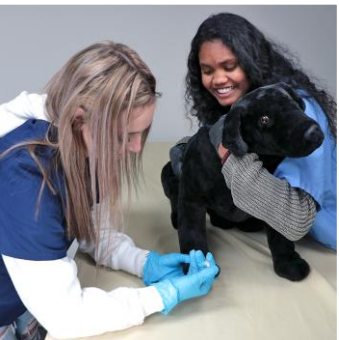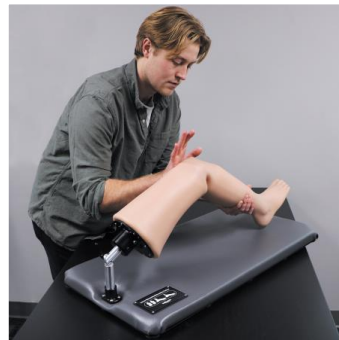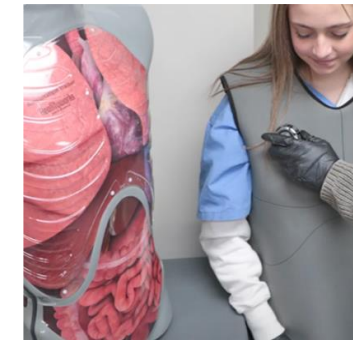

We partner with educators, administrators and industry leaders to ensure our offerings meet their needs and make lessons truly meaningful.

Realityworks®
Live it. Learn it.™
100% EMPLOYEE-OWNED



Presented by

Molly Kellgren

Health Science Education Product Manager

Realityworks[®]
Live it. Learn it.[™]

100% EMPLOYEE-OWNED

Attendees will:

- See a demonstration of the Bariatric Nursing Manikin
- Explore included curricula including lessons, activities, assessments
- Receive free resources!



Bariatric Nursing Manikin Curriculum

- Ambulation assistance
- Application of stockings
- Bathing patients with obesity
- Feeding
- Oral care
- Peri-care for patients with obesity
- Preparing to care
- Patient transfer and lifting
- Pressure injuries –
Considerations for patients with obesity
- Range of motion -
Considerations for patients with obesity
- Skin conditions in patients with obesity

Bariatric Nursing Manikin Activities



Assessments

Range of Motion Curriculum

Name: _____ Class: _____

Range of Motion (Focus on Empathy)

READ the following:

You are in a long-term care center due to a stroke. You have lost function and sensation of the right side of your body. You also have obesity issues which add to the difficulty moving. You can communicate but your speech is slurred. The television is on a news channel. The nursing assistant knocks and enters the room. He says "Hi, my name is Greg." He proceeds to uncover your arm and starts to exercise it. He continues to move your right extremities as he watches the news. He doesn't say anything to you and rarely looks at you. He seems distracted and in a rush to complete the exercises.

REFLECT on the situation

How do you feel while Greg is exercising you?

What could Greg say to make the experience more enjoyable and patient-focused?

SHARE your thoughts with a peer

Range of Motion Curriculum

Range of Motion Check-Off Instructor Resource

PREPARATION:

Supplies:

- Bath blanket
- Hand sanitizer or access to soap/water

After demonstration, have participants practice passive Range of Motion on the models using the following steps:

Optional: Evaluate the participant's ability to complete Range of Motion using the following steps. May score via points or satisfactory/unsatisfactory.


1.	Gather supplies needed (bath blanket)	
2.	Knock, enter room, and provide for privacy	
3.	Identify yourself with name and title (nurse aide, etc.)	
4.	Wash your hands with sanitizer or soap/water	
5.	Identify client by asking name and date of birth; verify this information	
6.	Explain the procedure to the client and ask if they have any pain or limitations	
7.	Position client in supine or sitting position	
8.	Neck ROM • Flexion: Bring chin to chest • Extension: Return head to upright position • Lateral flexion: Tilt head to each side • Rotation: Turn head in a circular movement	
9.	Shoulder ROM • Flexion: Raise arm forward to above head • Extension: Return arm to side • Abduction: Raise arm from side to above head with palm away from head • Adduction: Return arm to side ◦ May go past side to center of body if able • External rotation: Move arm to 90° from body; with elbow flexed, move lower arm upward, lateral to head • Internal rotation: Return lower arm to a downward position • Circumduction: Move arm in full circle	
10.	Elbow ROM • Flexion: Bend elbow upwards, bringing palm to shoulder • Extension: Straighten elbow thus lowering hand	
11.	Forearm ROM • Supination: Turn lower arm so palm is up • Pronation: Turn lower arm so palm is down	



Bariatric Nursing Manikin Demonstration



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Demo





Free Resources!



Free Resources

- Free lessons plans: [Free Lesson Plans | Realityworks | Innovative Learning Tools](#)
- Free posters and infographics: [Free Posters | Realityworks | Innovative Learning Tools](#)
- Blog: [Get Realistic Practice Caring for Bariatric Patients with This Manikin](#)



Let's connect!

We can help you start or refresh your own program!

Scan to:

- Connect with your local Realityworks account manager
- Request a free quote
- Ask questions and more!

Scan me)

Realityworks[®]
Live it. Learn it.™

100% EMPLOYEE-OWNED



Questions?

