

Station Setup Checklist

Materials:

- Geriatric Sensory Impairment Kit
- Geriatric Medication Management Simulation Kit
- Geriatric Simulator
- Visual Impairment Glasses
- Empathy Stations PDF (printed)
- Paper and pens
- Book or bag
- Magazines
- A chair
- A ball

Hearing Impairment Station (2 students)

- ☐ Set up the Hearing Impairment Simulator from the Geriatric Sensory Impairment Kit at a table for students to use
- ☐ Print the Hearing Impairment Station page from the Empathy Stations PDF and have it available for students

Arthritis Station (2 students)

- ☐ Set up the Geriatric Arthritis Simulator from the Geriatric Sensory Impairment Kit at a table for students to use
- ☐ Print the Arthritis Station page from the Empathy Stations PDF and have it available for students
- ☐ Provide a paper and pens for students to write with
- ☐ Provide a bag or book for students to pick up

Visual Impairment Station (Up to 10 students)

- ☐ Set up the Visual Impairment Glasses at a table for students to use
- ☐ Print the Visual Impairment Station page from the Empathy Stations PDF and have it available for students
- ☐ Provide up to 10 magazines for students to read
- ☐ Provide up to 10 pill bottles from the Geriatric Medication Management Simulation Kit for students to read

Medication Management Station (2 students per simulation kit)

- ☐ Set up the Medication Management Simulation Kits at a table for students to use
- ☐ Print the Medication Management Station page from the Empathy Stations PDF and have it available for students

Declining Mobility Station (3 students)

- ☐ Lay the Geriatric Simulator and its Quick Start Guide out at a table for students to use
- ☐ Print the Declining Mobility Station page from the Empathy Stations PDF and have it available for students
- ☐ Provide pens or pencils for students to pick up
- ☐ Provide a chair for students to sit in and stand up from
- ☐ Provide a ball for students to toss and catch